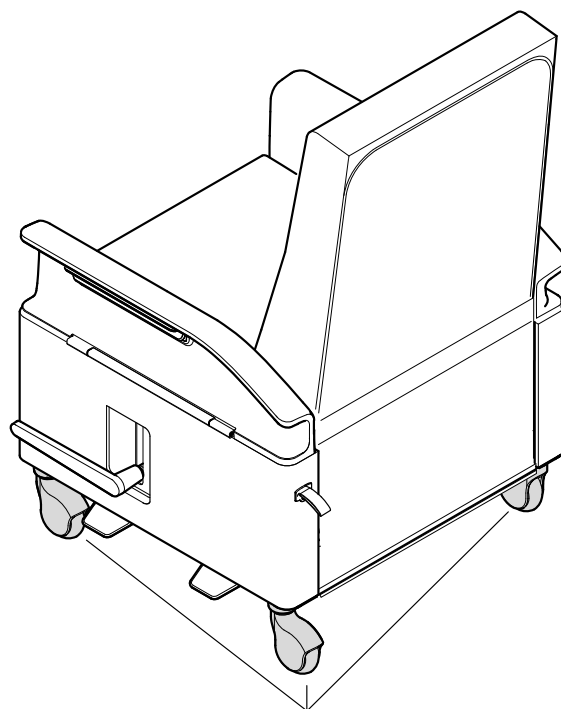
Retrofit Brake Activation Assembly on Empath™ Recliner

Locking Casters

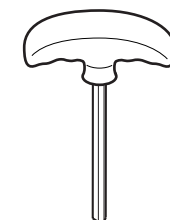
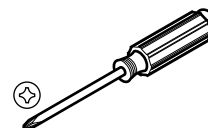
If you have a problem, question, or request, call
your local dealer, or Steelcase Line 1 at
888.STEELCASE (888.783.3522)
for immediate action by people who want to help you.

(Outside the U.S.A., Canada, Mexico, Puerto Rico,
and the U.S. Virgin Islands, call: 1.616.247.2500)



Or visit our website: www.steelcase.com

© 2015 Steelcase Inc.
Grand Rapids, MI 49501
U.S.A.


Printed in U.S.A.

1/8" & 5/32"
ALLEN WRENCH

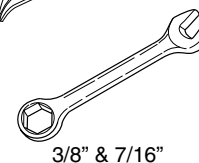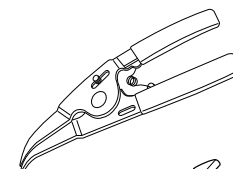3/8" & 7/16"

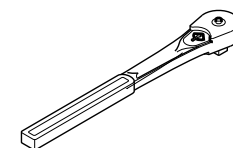7/16"
SOCKET

English - Pages 1 to 11 of 33
Spanish - Pages 12 to 22 of 33
Arabic - Pages 23 to 33 of 33
939599291 Rev D

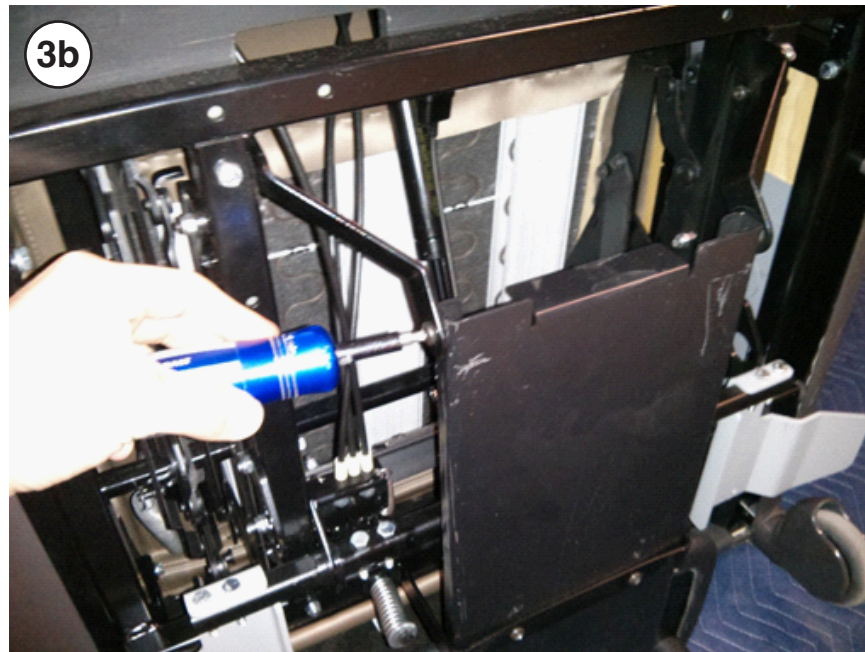
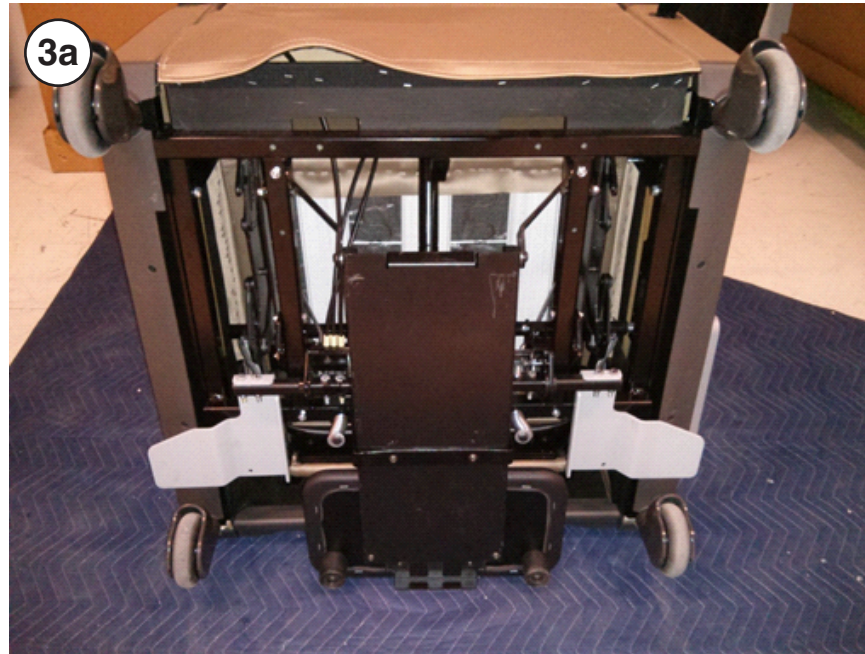
1. If the recliner has trendelenburg, activate the pedal and tilt the recliner back, then release the pedal.

2. Use a blanket to protect fabric. Place the recliner on its front as shown.



3a. If recliner is equipped with a foot tray, remove the foot tray per steps 3b & 3c. If no foot tray, go to step 4.

3b. Using a phillips screwdriver, remove the two (2) screws holding the back of the foot tray.


3c. Using a 5/32" allen wrench, remove the two (2) central bolts in the foot tray. Remove the foot tray from the recliner.

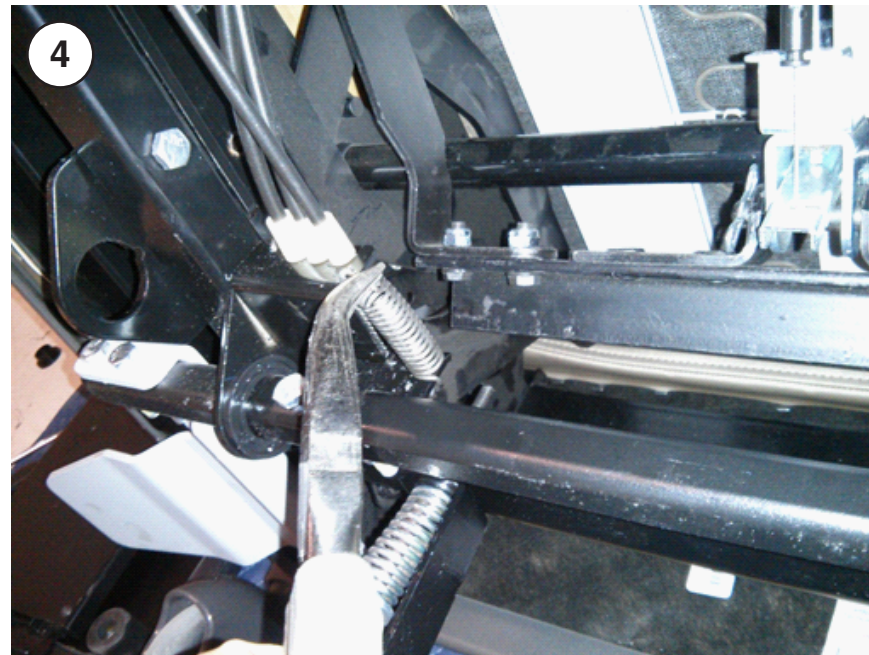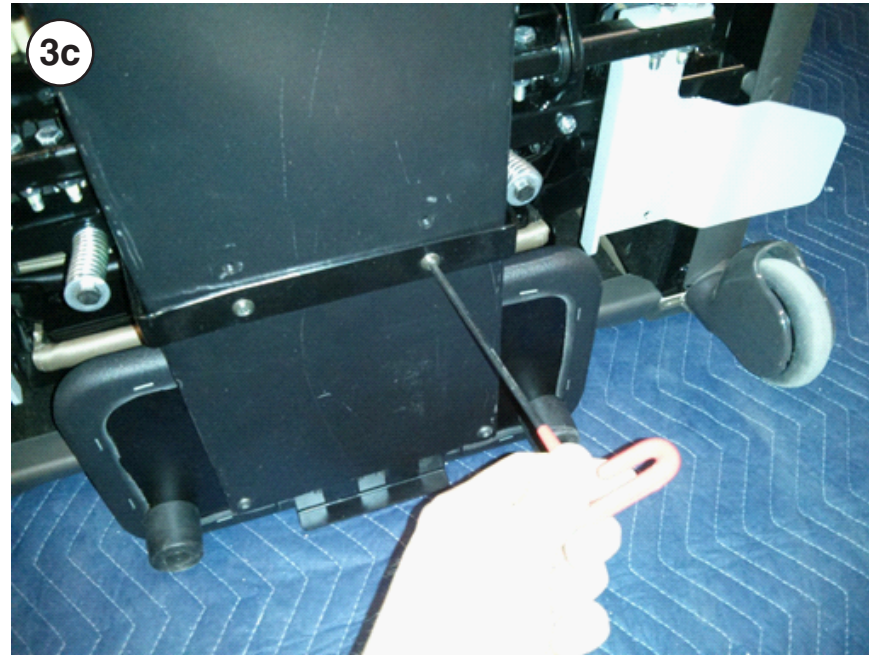
4. Using needle nose pliers, remove and replace the two (2) extension springs.

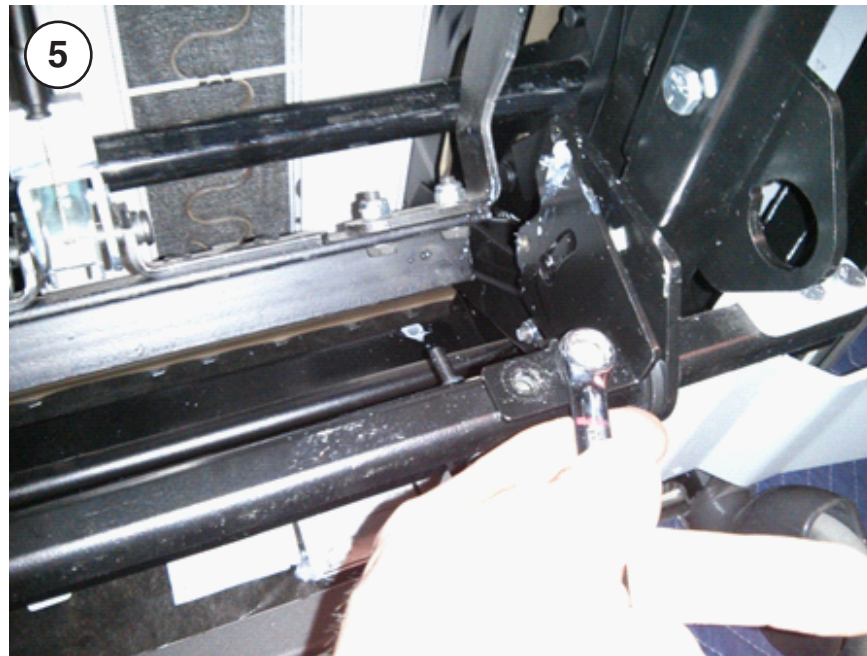- 5.** Remove two (2) bolts holding on upright bracket using 7/16" wrenches. Remove bracket.

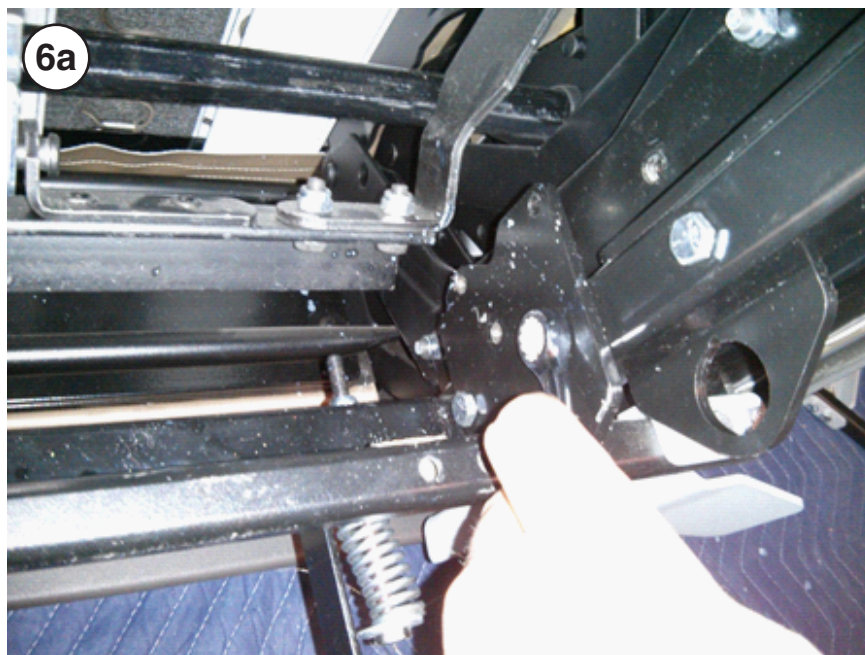
6. For units without trendelenburg, remove two (2) bolts holding on mounting plate (6a & 6b). For trendelenburg units, skip to step 8.


7. Rotate mounting bracket down to access screws.

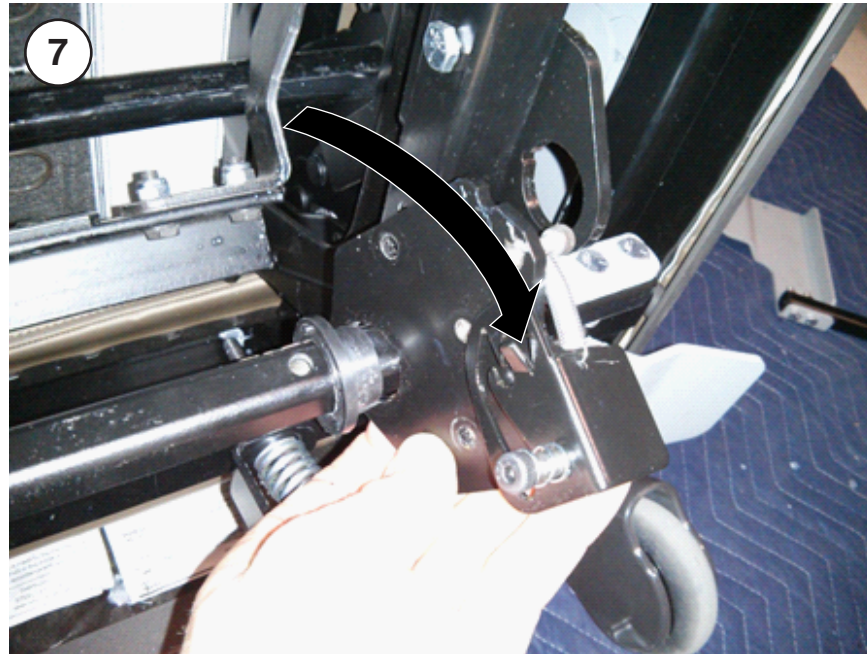8. Remove shoulder bolt with 1/8" allen wrench and 3/8" box end wrench. This can be accessed in place on a trendelenburg recliner.


9. Using a 5/32 allen wrench & a 3/8" box end wrench, remove the front screw. This can be accessed in place on a trendelenburg equipped recliner with the mechanism tilted back.

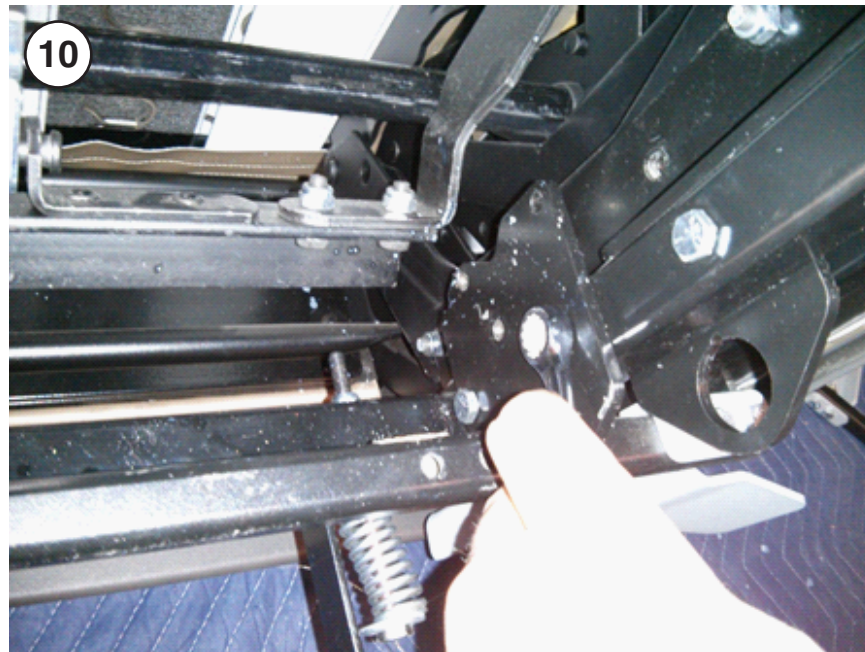10. If bracket has been unattached, replace and tighten two (2) bolts and nuts.


11. Install new upright bracket (21699901) using two (2) bolts and nuts with 7/16" wrench.

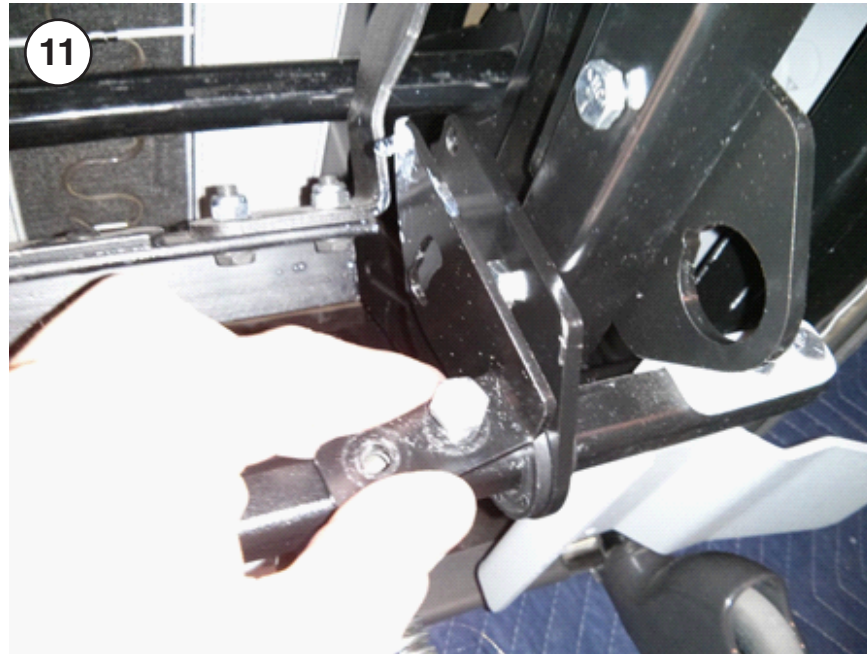12. Spray pin with white lithium grease.

13. Spray interior of lock actuation plate with white lithium grease.

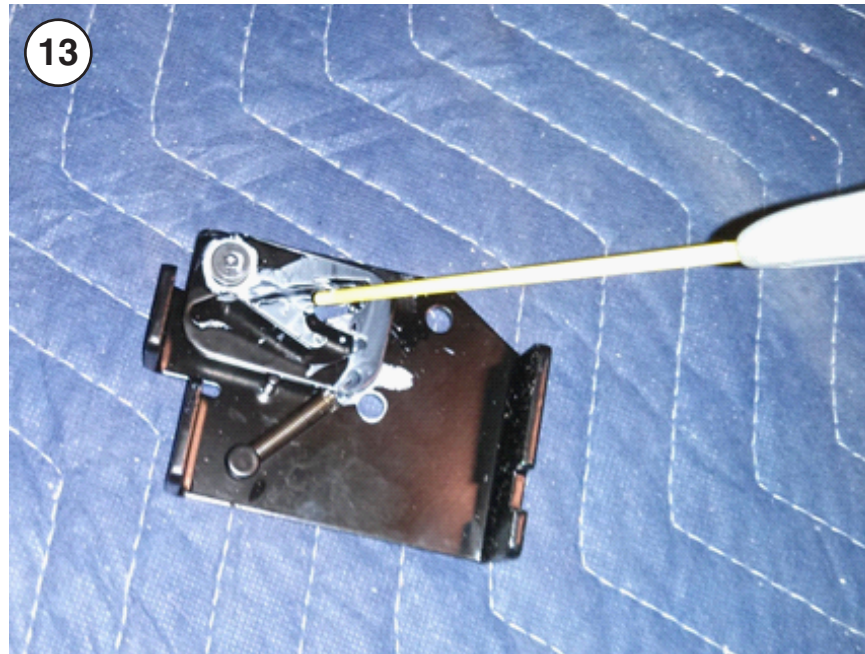
14. Place brake activation assembly on mounting plate and hand tighten two (2) shoulder screws.

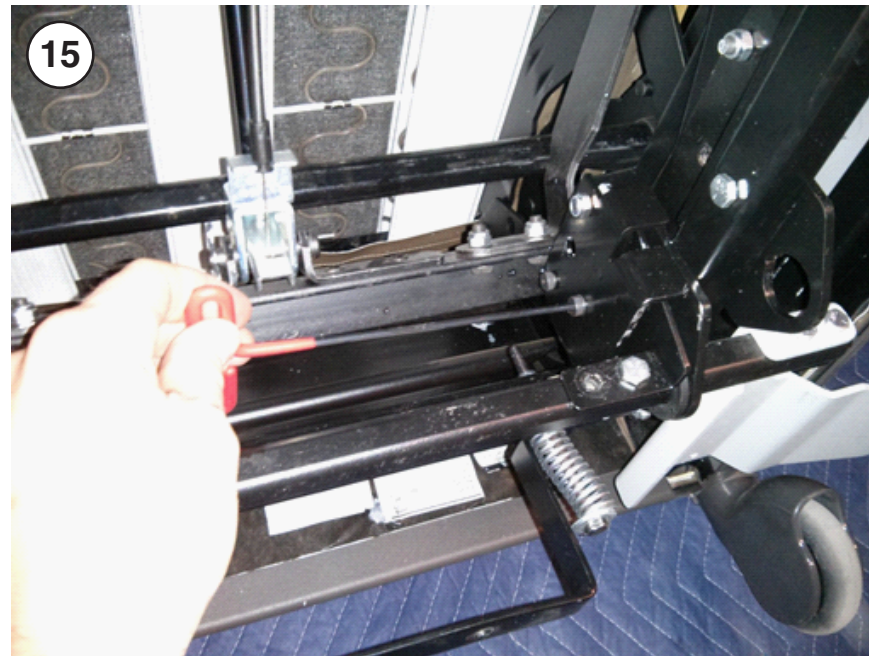15. Tighten down both shoulder screws using 1/8" allen wrench.

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