

Mineral™ & Mitra® Folding, Full, Control Arm & Mechanism Replacement

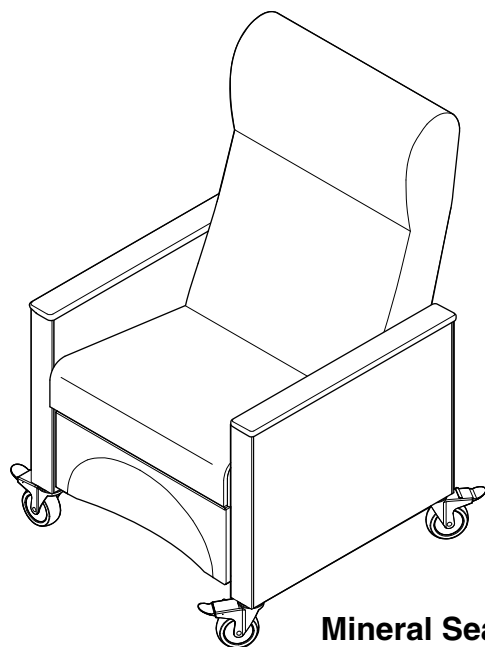Mineral Seating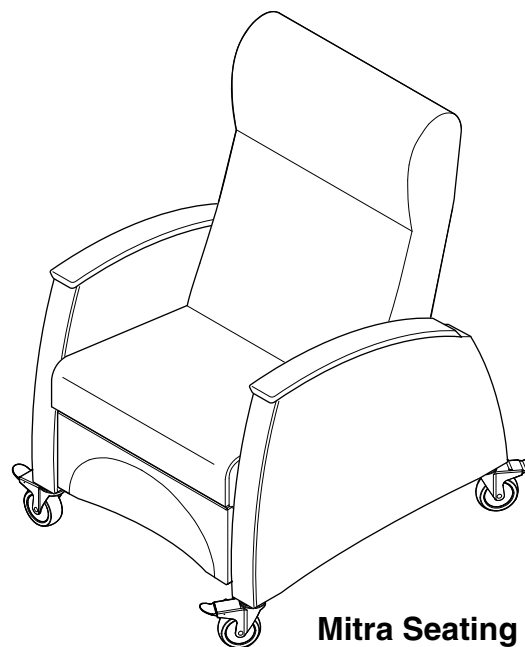

Mitra Seating

If you have a problem, question, or request, call
your local dealer, or Steelcase Line 1 at
888.STEELCASE (888.783.3522)
for immediate action by people who want to help you.



(Outside the U.S.A., Canada, Mexico, Puerto Rico,
and the U.S. Virgin Islands, call: 1.616.247.2500)

Or visit our website: www.steelcase.com


© 2015 Steelcase Inc.
Grand Rapids, MI 49501
U.S.A.

Printed in U.S.A.


Replace Mechanism - Pages 2 through 7.
Replace Control Arm - Pages 3 and 4.
Replace Full or Folding Arm - Page 5.

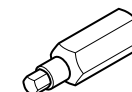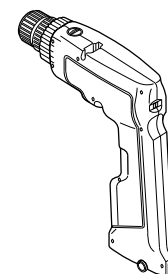HEX DRIVE


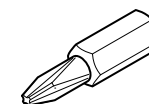
PHILLIPS DRIVE

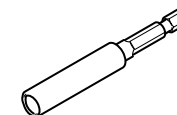English - Pages 1 to 7 of 21
Spanish - Pages 8 to 14 of 21
Arabic - Pages 15 to 21 of 21
939599278 Rev E

Remove Seat Back

1. Remove thumb screw.
2. Pull upwards to remove seat back.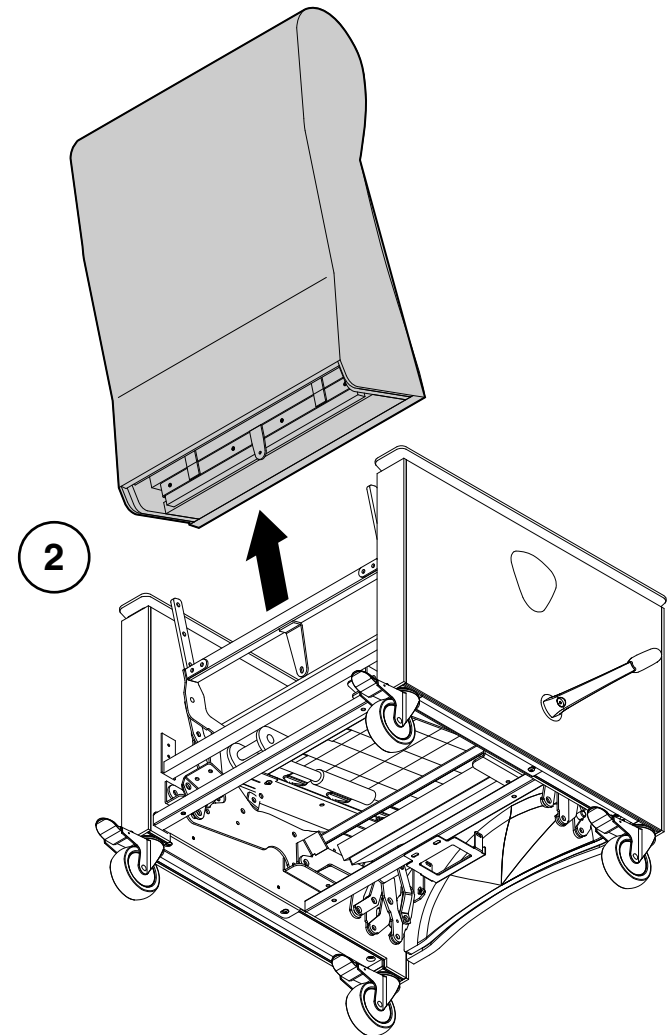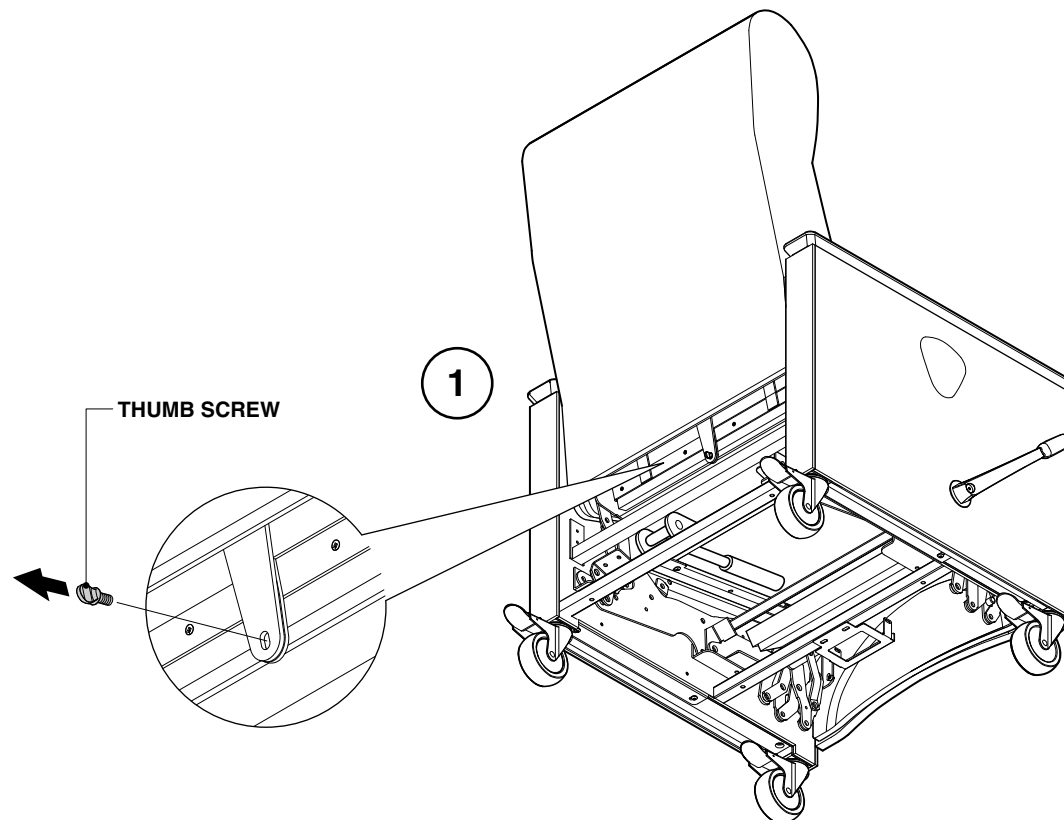


Remove Control Arm


1. Remove cable from mechanism.

CAUTION: Be careful not to push lever in direction "B" to avoid actuating mechanism.


2. Disconnect cabling for Heat/Massage unit, if available.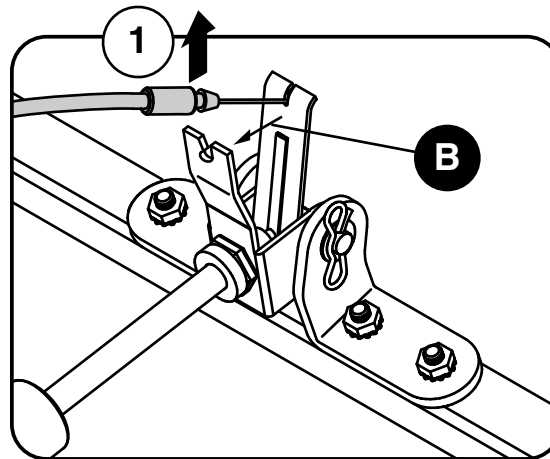

After March 19, 2011

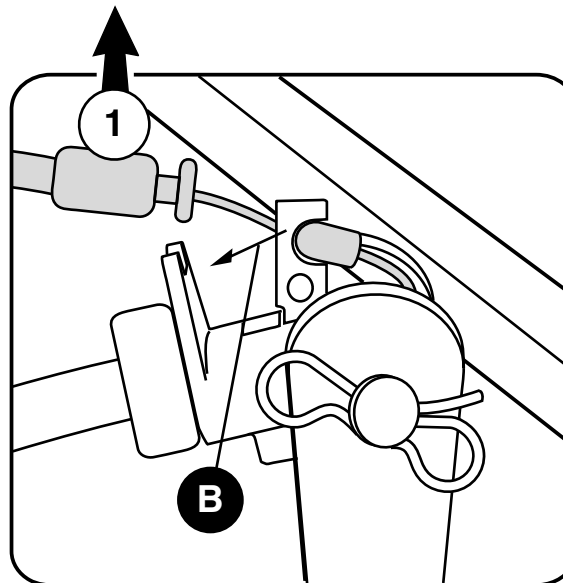Before March 19, 2011





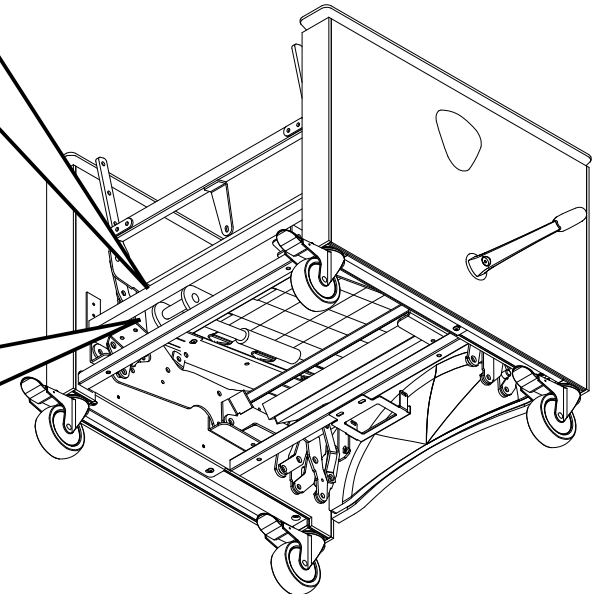
Remove Control Arm (cont.)

3. Remove screws to remove the handle assembly.

NOTE: For recliners purchased before October 17, 2011, the new cable supplied with the replacement mechanism must be used.

- Remove the old cable from the handle assembly on the recliner and discard the old cable.
- Remove new handle assembly from the supplied cable and discard the new handle assembly.
- Attach the new cable to the old handle assembly.



4. Remove screw and arm lever plate to remove the footrest handle.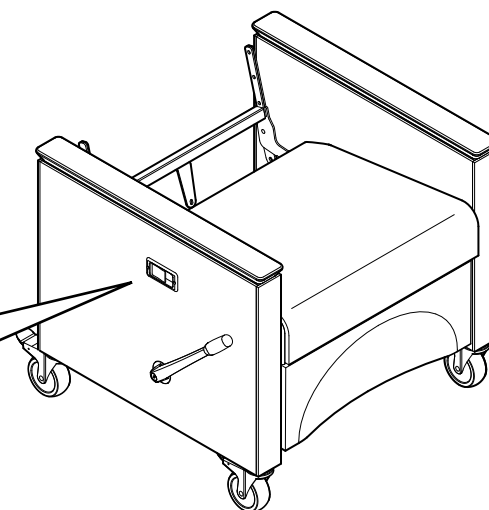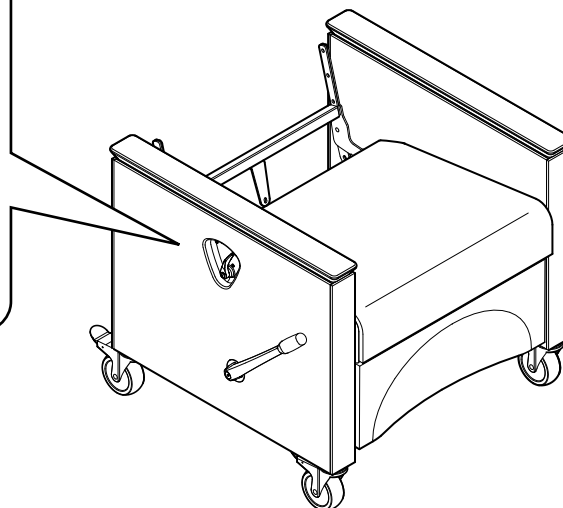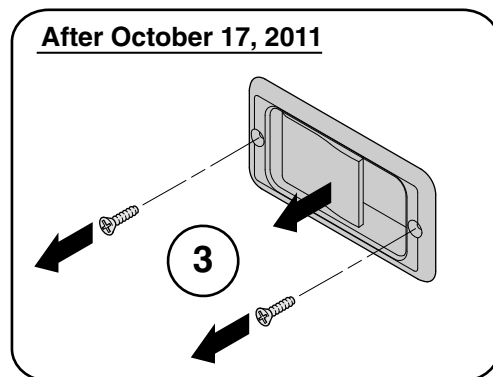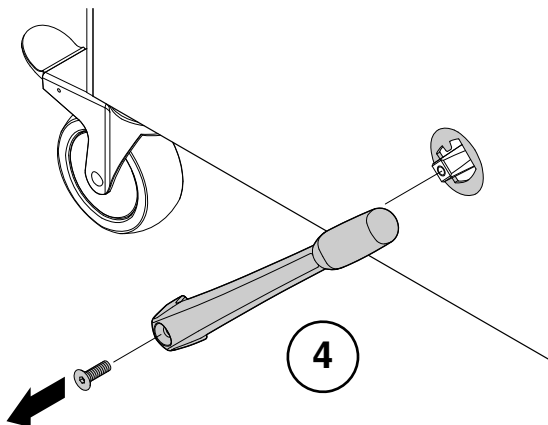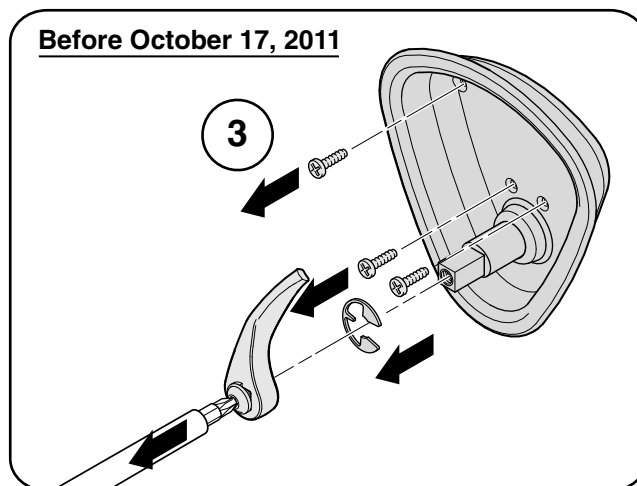

Remove Control Arm (cont.) and Full or Folding Arm


5. Remove two (2) button head cap screws on each side (5a) and remove crossbar (5b).

6. Remove four (4) machine screws (6a) and remove arm (6b). Repeat for opposite side.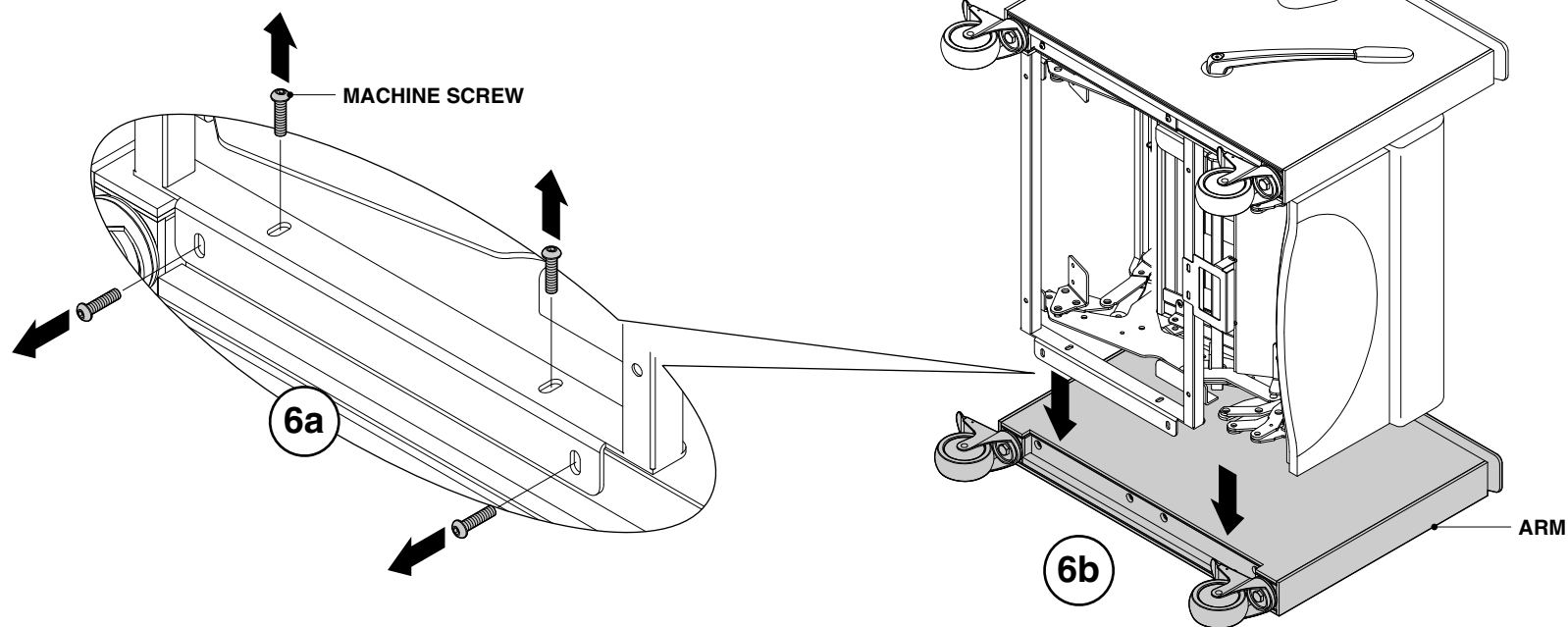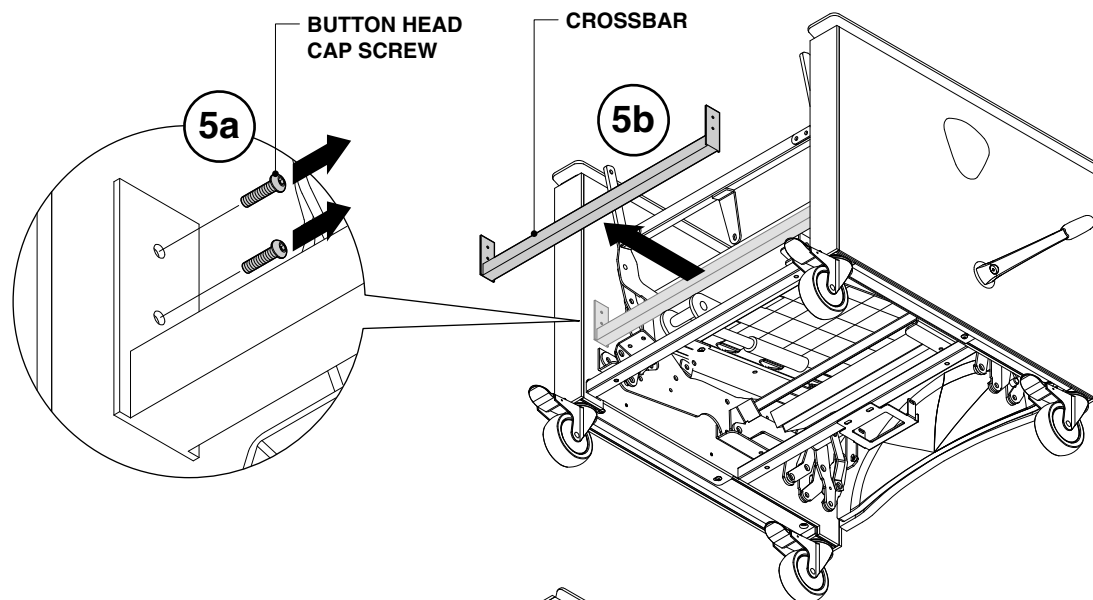

Remove Outer & Inner Footrest


1. Remove four (4) button head cap screws (1a) and remove outer footrest (1b).
2. Remove four (4) button head cap screws (2a) and remove inner footrest (2b).

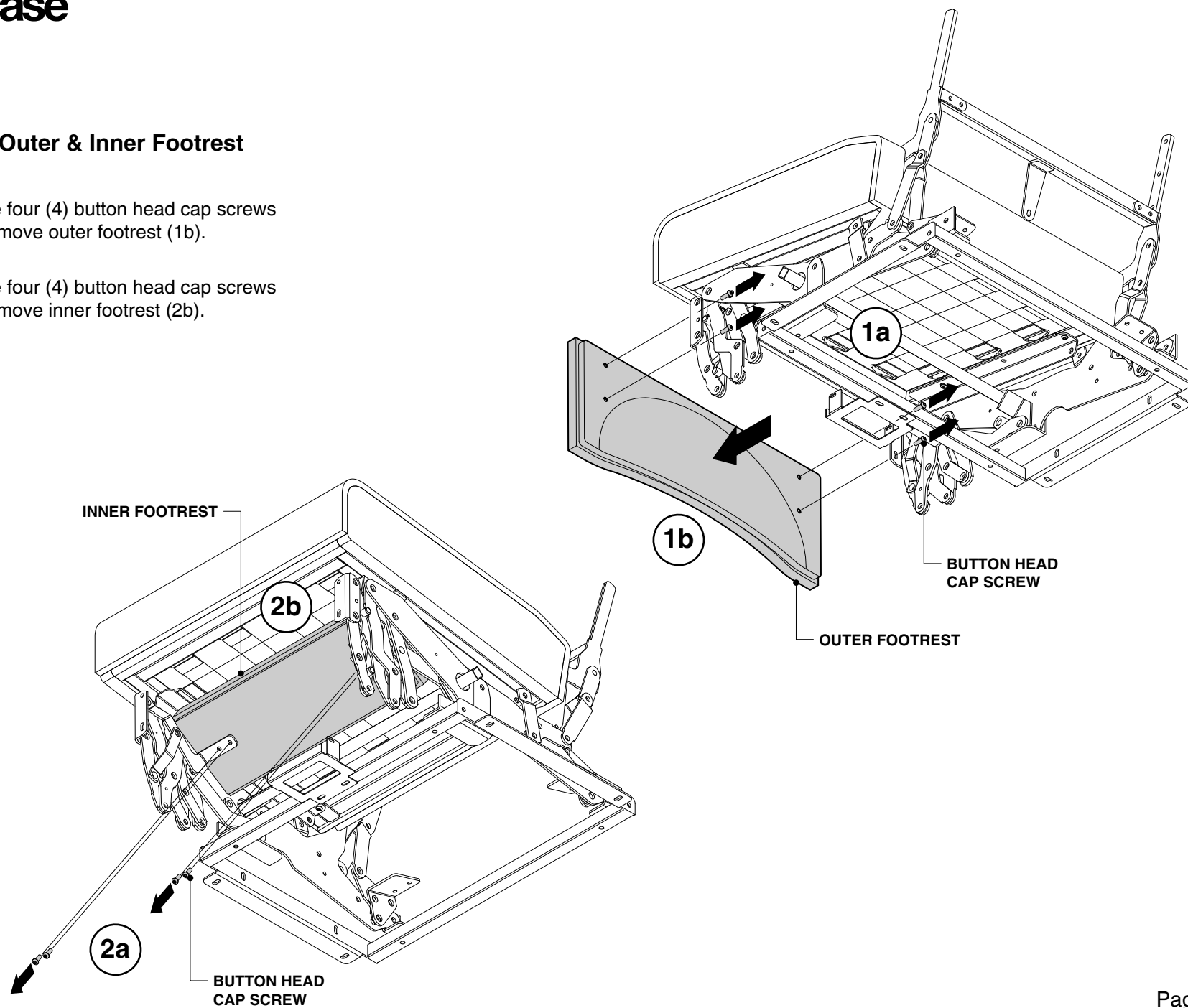Remove Seat Cushion

1. Remove four (4) button head cap screws (1a) and remove seat cushion (1b).

Reverse steps to install new component(s) to install new mechanism.

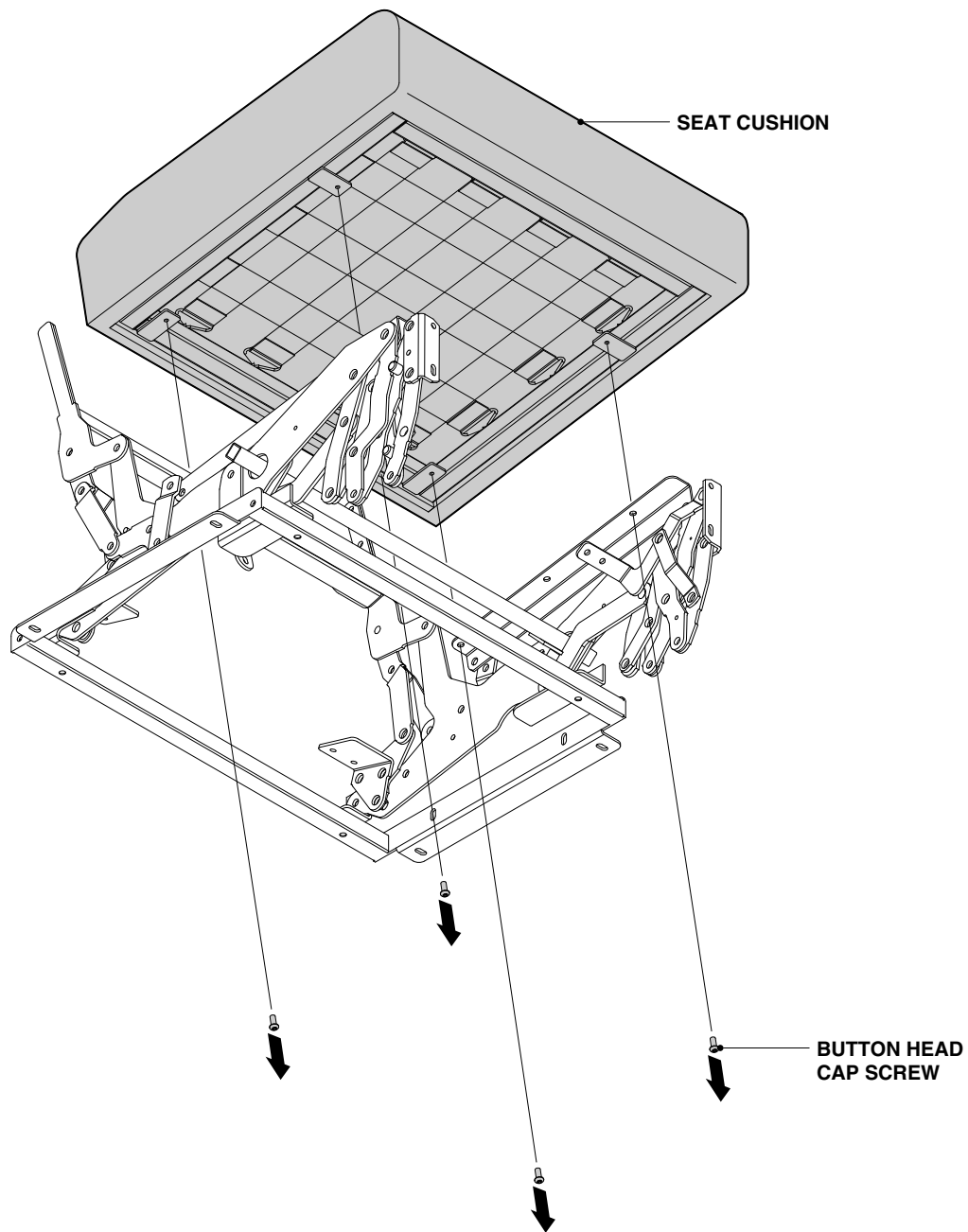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