

Assembly Instructions ■

600 & 800 Series Stool

Adjustable-Height Legs

January 2023

Adjustable-Height Leg Assembly

1. Remove adjustable-height legs from the parts bag. Using a #3 Phillips screw driver, unscrew the $\frac{1}{4}$ -20 x $\frac{5}{8}$ " screws with lock washers from each adjustable-height leg (Figure 1).
2. Carefully turn stool upside down on a soft protective surface. Slide each adjustable-height leg into the leg tube of the stool frame and align the mounting holes to the desired stool height. Using a #3 Phillips screw driver, thread each screw through the mounting hole in the leg tube, and into the adjustable-height leg until the lock washer fits snugly. Tighten all screws securely (Figure 1).

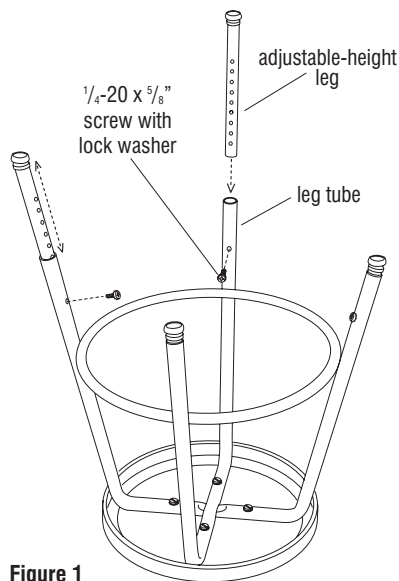


Figure 1

