Lasata recliner

Operating instructions

carolina
an OFS company

carolina.ofs.com

800.763.0212

10.24

Lasata reclinable del paciente

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carolina
an OFS company

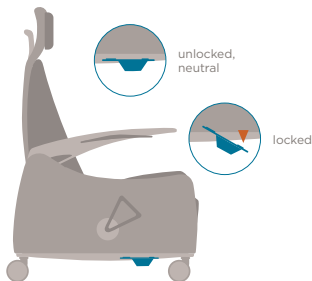
carolina.ofs.com

800.763.0212

10.24


Dual sided central locking caster pedals

- With foot, apply pressure on the red lock symbol. By design, only the right front-facing, and left rear-facing casters lock securing the recliner in a locked position
- With foot, apply pressure on the green unlock symbol. The casters will unlock when in the neutral, level position of the pedal

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
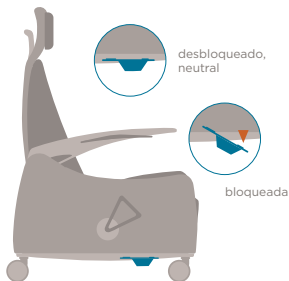
Foot ottoman handle

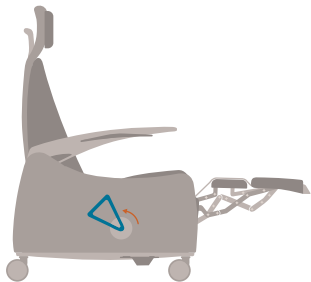- Using your hand, grip the foot ottoman handle and rotate from front to back of the recliner for foot ottoman to extend
- Using your hand, grip the foot ottoman handle and rotate from back to front of the recliner to close the foot ottoman

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
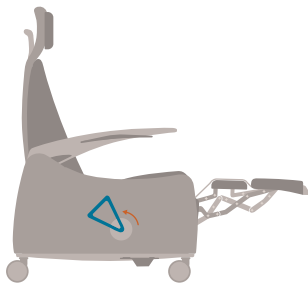
Dual back recliner activation paddles

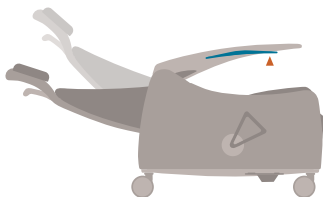- Located under each arm, press the paddles for back to activate and recline
- Located under each arm, press the paddles for back to activate to return to upright position

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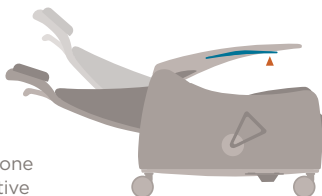
Dual sided trendelenburg pedal

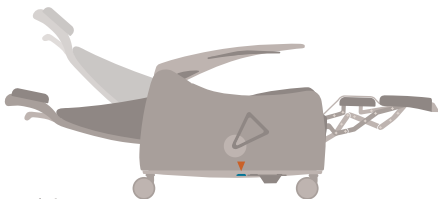
- Begin with the unit fully reclined and the foot ottoman extended. To activate, step on trendelenburg foot pedal (heart) while applying pressure on the recliner's back to lock into Trendelenburg position.
- To return recliner to its normal reclined position press down on trendelenburg pedal and recliner automatically return to normal position

Trendelenburg is not offered on the bariatric models

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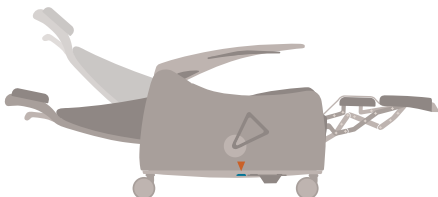
Dual sided pivoting arm lock

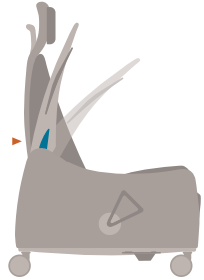
- For the arm to pivot up, simply lift the lever upward while lifting the arm for raised position.
- For the arm to pivot down, simply lift the lever upward while lowering the arm for down position.
- Once placed in the fully raised or fully lowered position, the arm will automatically lock.

Dual sided pivoting arms is not offered on bariatric models, they remain stationary. All models are also available as stationary arms without pivoting lever.

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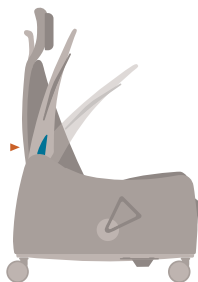Care and maintenance

Carolina products are manufactured with careful attention to detail and with an on-going commitment to achieve a level of quality that is free of defects in materials and workmanship, given normal use and proper care.

Carolina offers an extensive program of graded-in textiles. Regular cleaning of the chair will enhance longevity of the product.

Activate chair mechanisms periodically to ensure proper function of moving parts.

Lasata bariatric and orthopedic models are available.

Safety information

- Do not sit on arms
- Do not sit on extended foot ottoman
- Close the foot ottoman when not in use
- Do not use recliner as patient transport
- Do not lift the chair by the outside back
- Keep casters locked when not in use
- Do not remove this tag

For more information refer to warranty, terms and conditions on our website.

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